



# VetScan® Canine Ehrlichia Rapid Test



For the qualitative detection of antibodies to *E. canis*, *E. chaffeensis*, and/or *E. ewingii* in canines.

## Accurate. Easy. Affordable. It's a singularly better approach to rapids.

The Abaxis Canine Ehrlichia Rapid Test is a highly sensitive and specific test for the detection of antibodies to *E. canis*, *E. chaffeensis*, and/or *E. ewingii* in canine whole blood, serum, or plasma. The Abaxis Canine Ehrlichia Rapid Test is the first and only single rapid test for canine Ehrlichia antibodies and offers the proven VetScan advantages of speed, simplicity, and value.

- Cost effective
- Easy to read results
- Easy to use—just add sample and buffer
- Whole blood, serum, or plasma
- Fast - results in 8-10 minutes
- Room temperature storage
- Available in a 25-test kit
- **Only single test available** for canine Ehrlichia antibodies

## Clinically Validated Confidence

The Abaxis Canine Ehrlichia Rapid Test delivers quality that is equal to or better than other in-clinic lateral flow devices.

Sensitivity 93.4%; n=212

Specificity 96.7%; n=214

Using *E. canis*/*E. chaffeensis*  
IFA titer reference method

*E. canis* Sensitivity 97.8%; n=45

*E. chaffeensis* Sensitivity 95.0%; n=40

*E. ewingii* Sensitivity 97.9%; n=47

## Clinical Signs

### *E. canis*

- Fever, anorexia, weight loss
- Pale mucous membranes, bleeding disorders
- Splenomegaly, Lymphadenopathy
- Uveitis and retinal lesions
- CNS signs

### *E. chaffeensis*

- Epistaxis, vomiting
- Lymphadenopathy

### *E. ewingii*

- Fever, anorexia
- Joint swelling - polyarthritis
- CNS signs



800.822.2947



[www.abaxis.com](http://www.abaxis.com)



[vetscan@abaxis.com](mailto:vetscan@abaxis.com)

Singularly **Better.**




**ABAXIS**  
GLOBAL DIAGNOSTICS

Abaxis, Inc.  
Worldwide Headquarters


3240 Whipple Road  
Union City, CA 94587  
Tel 800 822 2947  
Fax 510 441 6150

[www.abaxis.com](http://www.abaxis.com)


# Ehrlichia Test Instructions

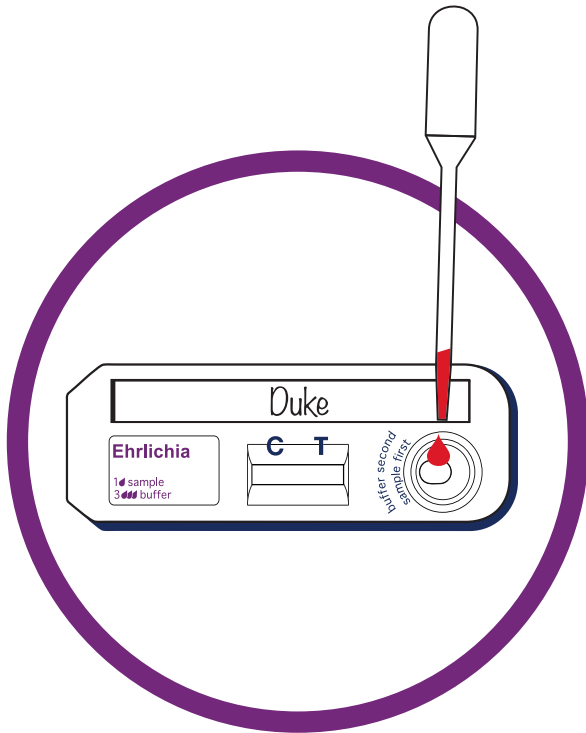

**1** Add 1 drop of blood, serum or plasma to the sample well and wait 30-60 seconds.

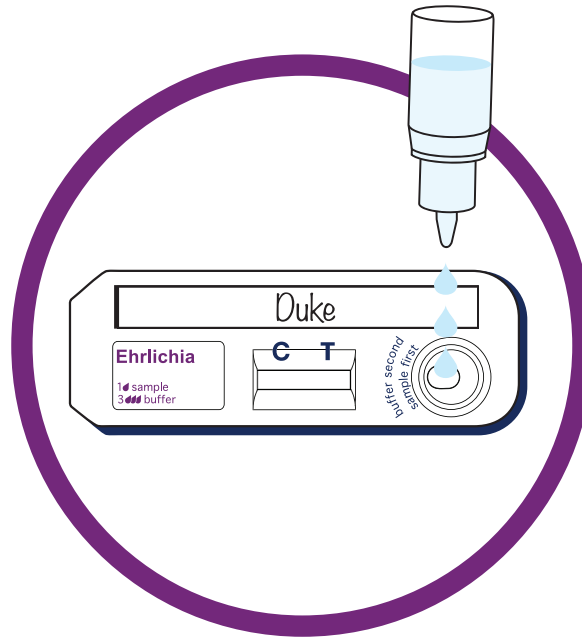

**2** Add 3 drops of Chase Buffer to the sample well.

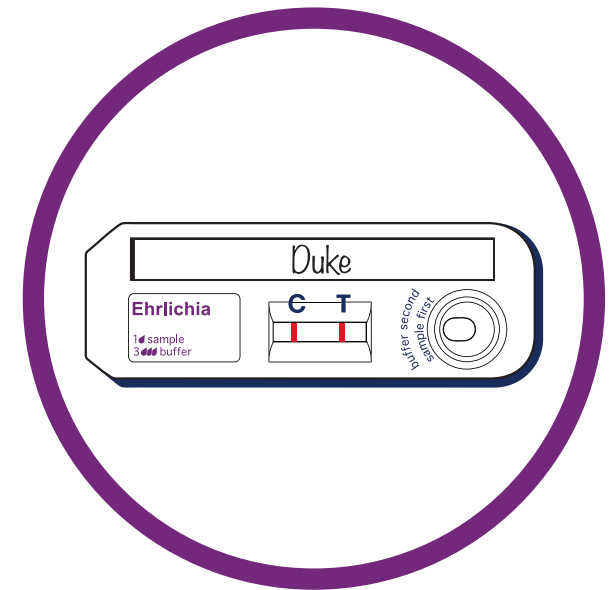

**3** Read results within 8 to 10 minutes. **Positive Example**



800.822.2947



[www.abaxis.com](http://www.abaxis.com)



[vetscan@abaxis.com](mailto:vetscan@abaxis.com)

Singularly **Better.**



Abaxis, Inc.  
Worldwide Headquarters

3240 Whipple Road  
Union City, CA 94587  
Tel 800 822 2947  
Fax 510 441 6150

[www.abaxis.com](http://www.abaxis.com)