



VetScan® Canine Lyme Rapid Test



For the qualitative detection of antibodies to *Borrelia burgdorferi* in canines.

Accurate. Easy. Affordable. It's a singularly better approach to rapids.

The VetScan Canine Lyme Rapid Test is a highly sensitive and specific test for the detection of antibodies to *Borrelia burgdorferi* in canine whole blood, serum or plasma. The VetScan Canine Lyme Rapid Test is the first and only single rapid test for canine Lyme disease and offers the proven VetScan advantages of speed, simplicity and value.

- Cost-effective
- Easy to read results
- Easy to use - just add sample and buffer
- Whole blood, serum, or plasma
- Fast - results in 8-10 minutes
- Room temperature storage
- Available in a 25-test kit
- **Only single test** available for canine Lyme disease

Clinically Validated Confidence

The VetScan Canine Lyme Rapid Test delivers quality that is equal to or better than other in-clinic lateral flow devices.

Sensitivity 100%; n=100

Specificity 100%; n=50

Using validated ELISA reference method

Clinical Signs

- Acute, chronic or shifting lameness
- Stiffness
- Sensitivity to touch
- Fever and lack of appetite
- Lymphadenopathy
- Glomerulonephritis is possible
- Cardiac and neurological diseases are rare



800.822.2947



www.abaxis.com



vetscan@abaxis.com

Singularly **Better.**




ABAXIS
GLOBAL DIAGNOSTICS

Abaxis, Inc.
Worldwide Headquarters


3240 Whipple Road
Union City, CA 94587
Tel 800 822 2947
Fax 510 441 6150

www.abaxis.com


Lyme Test Instructions

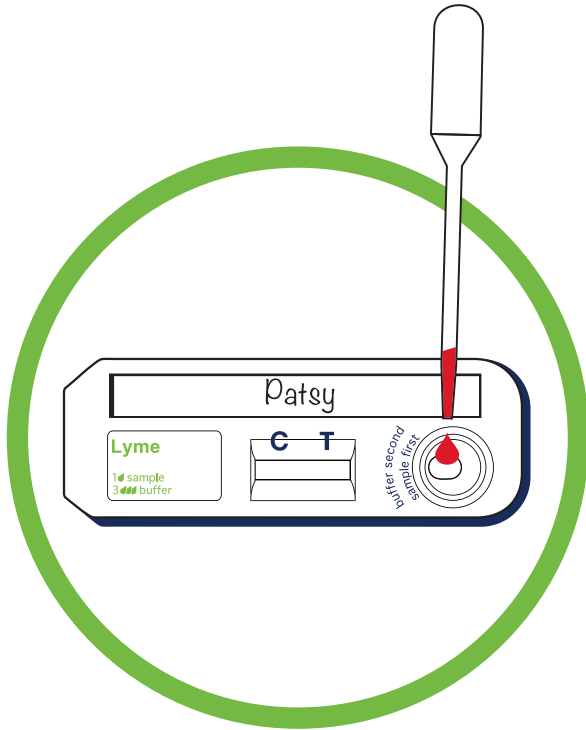

1 Add 1 drop of blood, serum or plasma to the sample well and wait 30-60 seconds.

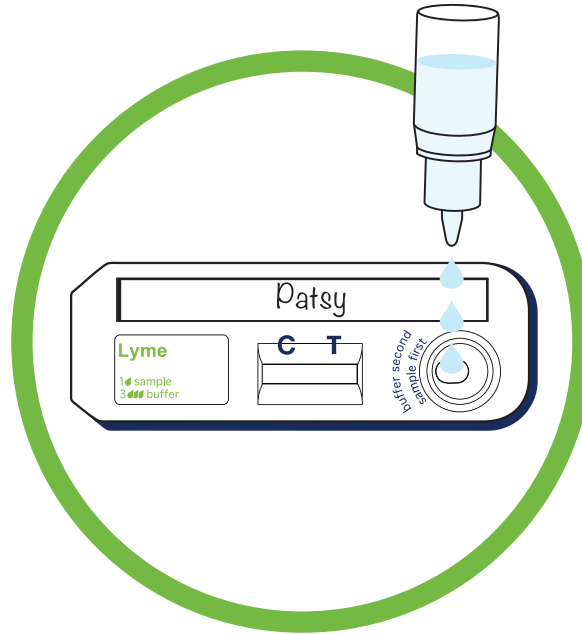

2 Add 3 drops of Chase Buffer to the sample well.

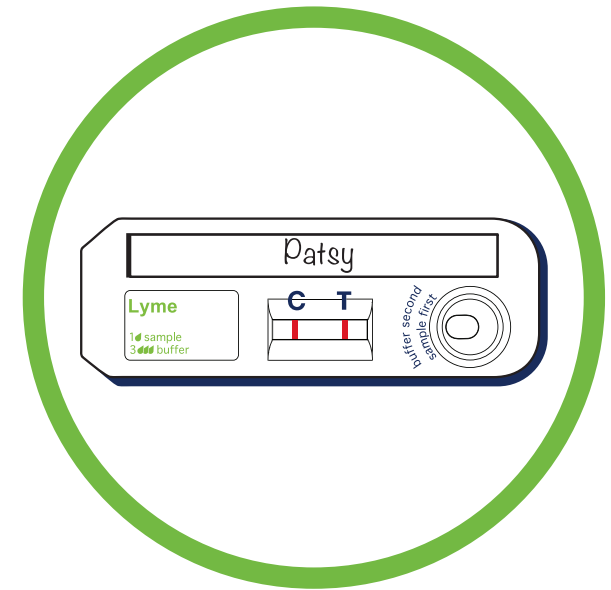

3 Read results within 8 to 10 minutes. **Positive Example**



800.822.2947



www.abaxis.com



vetscan@abaxis.com

Singularly **Better.**



ABAXIS
GLOBAL DIAGNOSTICS

Abaxis, Inc.
Worldwide Headquarters

3240 Whipple Road
Union City, CA 94587
Tel 800 822 2947
Fax 510 441 6150

www.abaxis.com