



# VetScan<sup>®</sup> Canine Heartworm Rapid Test

For the qualitative detection of *Dirofilaria immitis* antigen in canines and felines.



## Accurate. Easy. Affordable. It's a singularly better approach to rapids.

The VetScan Heartworm Rapid Test Kit is a highly sensitive and specific test for the detection of *Dirofilaria immitis* antigen in canine and feline whole blood, serum or plasma. The test delivers speed, simplicity and VetScan quality at a price point significantly lower than other brands.

- Cost effective
- Easy to read results
- Easy to use - just add sample and buffer
- Whole blood, serum or plasma
- Fast - results in 10 minutes
- Room temperature storage
- Available in 25-test or 100-test kit

### Clinical Signs

- Weight loss, lethargy
- Fainting
- Coughing
- Congestive heart failure

## Clinically Validated Confidence

VetScan Rapid Tests deliver quality that is equal to or better than other in-clinic lateral flow devices.

Canine Heartworm Data			Canine Heartworm Sensitivity				
Test Name	Sensitivity	Specificity	Female Worms	VetScan	Snap	Solo Step	Witness
VetScan	92%	100%	1-4	89%	89%	78%	67%
Snap <sup>®</sup>	92%	100%	1-10	85%	85%	69%	62%
Solo Step <sup>®</sup>	84%	100%	1-15	88%	88%	75%	69%
Witness <sup>®</sup>	84%	100%	1-25	90%	90%	81%	76%
			1-55	92%	92%	85%	81%

(compared to necropsy verified samples; n=49; (25 positive, 24 negative); source: data on file at Abaxis)

### Feline Heartworm Data

**Sensitivity 79.9%**

**Specificity 99.7%**

(compared to necropsy verified samples; n = 380; source: USDA submission)

n=380	Necropsy (+)	Necropsy (-)
VetScan Heartworm Rapid Test (+)	23	1
VetScan Heartworm Rapid Test (-)	6	350



800.822.2947



www.abaxis.com



vetscan@abaxis.com

Singularly **Better.**




Abaxis, Inc.  
Worldwide Headquarters


3240 Whipple Road  
Union City, CA 94587  
Tel 800 822 2947  
Fax 510 441 6150

www.abaxis.com


# Heartworm Test Instructions

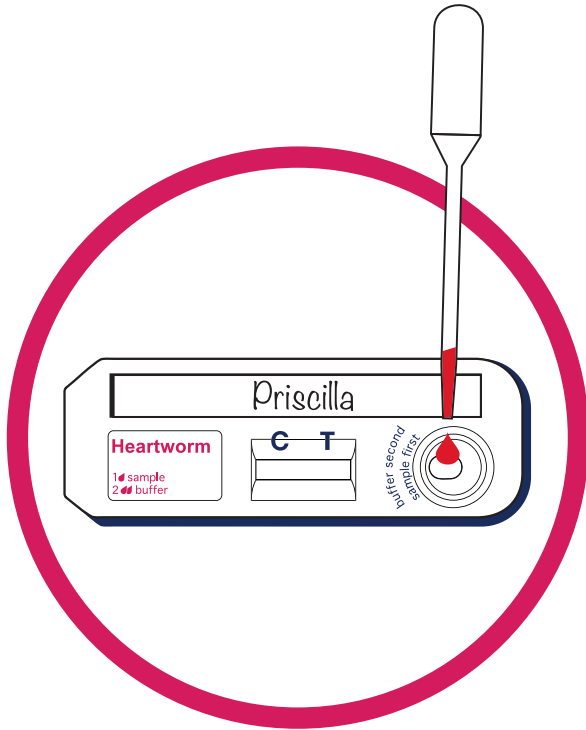

**1** Add 1 drop of blood, serum or plasma to the sample well and wait 3 - 5 seconds.

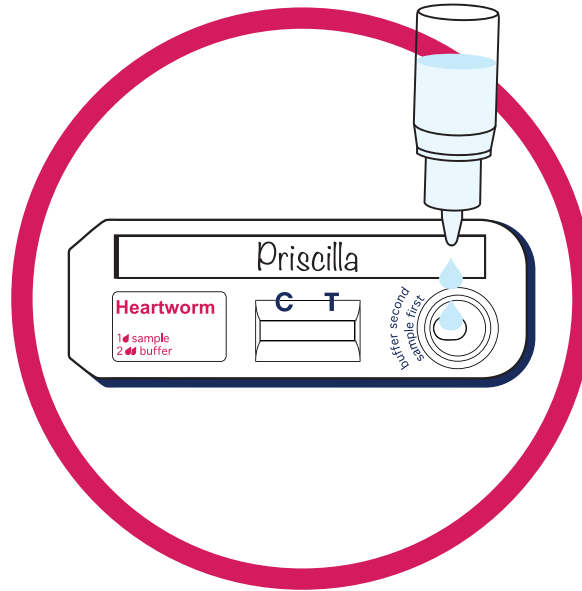

**2** Add 2 drops of Chase Buffer to the sample well.

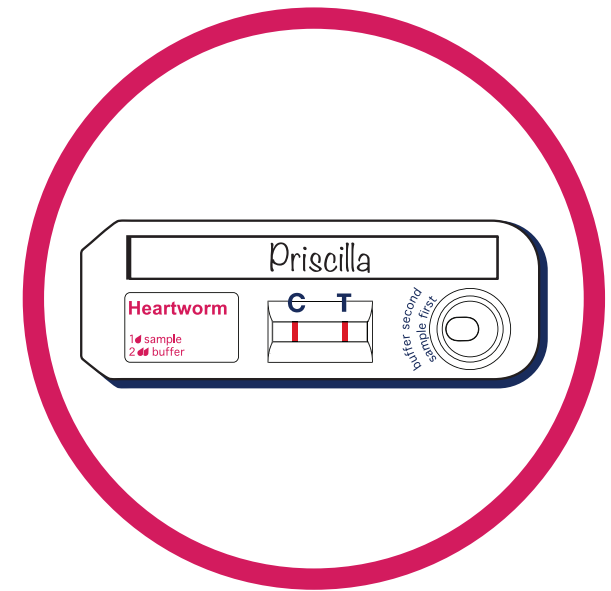

**3** Read results within 10 minutes.  
**Positive Example**



800.822.2947



[www.abaxis.com](http://www.abaxis.com)



[vetscan@abaxis.com](mailto:vetscan@abaxis.com)

Singularly **Better.**



Abaxis, Inc.  
Worldwide Headquarters

3240 Whipple Road  
Union City, CA 94587  
Tel 800 822 2947  
Fax 510 441 6150

[www.abaxis.com](http://www.abaxis.com)