



VetScan® FeLV/FIV Rapid Test



For the qualitative detection of Feline Leukemia Virus antigen and Feline Immunodeficiency Virus antibodies in feline whole blood, serum, or plasma.

Accurate. Easy. Affordable. Providing the flexibility you need in rapids.

The Abaxis Feline FeLV/FIV Rapid Test is a highly sensitive and specific test for the detection of antibodies to Feline Immunodeficiency Virus and Feline Leukemia Virus antigen in feline whole blood, serum, or plasma. The Abaxis Feline FeLV/ FIV Rapid Test offers the proven VetScan advantages of speed, simplicity, and value.

- Cost effective
- Easy to read results
- Easy to use—just add sample and buffer
- Whole blood, serum, or plasma
- Fast - results in 10 minutes
- Room temperature storage
- Available in a 25-test kit

FeLV Sensitivity 96%; n=93
Specificity 99%; n=140

FIV Sensitivity 99%; n=114
Specificity 97%; n=110

Agreement by IFA and FeLV/FIV commercial kit methodologies.

Clinical Signs

FeLV

May be none or non-specific:

- Depression
- Weakness
- Enlarged lymph nodes
- Anemia
- Lymphoma
- Blood in stool
- Chronic or recurring infections
- Weight loss
- Diarrhea
- Excessive drinking and urination

FIV

- May be none or signs of immune deficiency causing infections in any organ system, cancer, or neurologic issues.



800.822.2947



www.abaxis.com



vetscan@abaxis.com

Singularly **Better.**




ABAXIS
GLOBAL DIAGNOSTICS

Abaxis, Inc.
Worldwide Headquarters


3240 Whipple Road
Union City, CA 94587
Tel 800 822 2947
Fax 510 441 6150

www.abaxis.com


FeLV/FIV Test Instructions

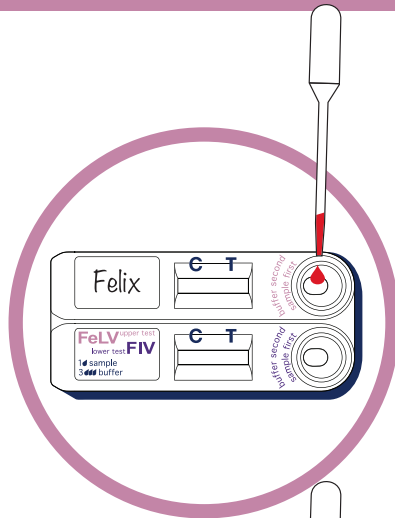


1
Add 1 drop of blood, serum or plasma to the FeLV sample well and wait 5 to 10 seconds.

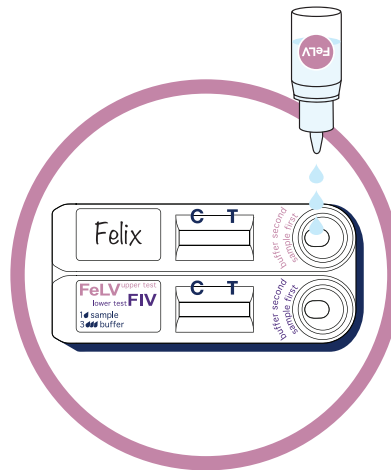


2
Add 3 drops of FeLV Chase Buffer (white cap) to the FeLV sample well.

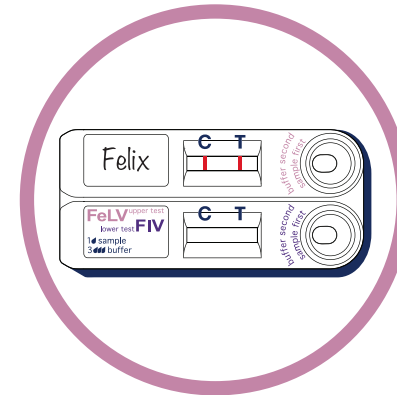


3
Read results within 10 minutes.
Positive Example

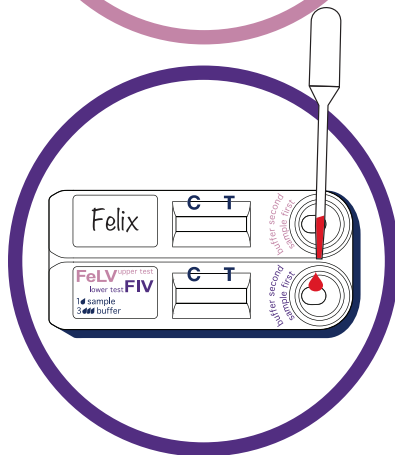

1
Add 1 drop of blood, serum or plasma to the FIV sample well and wait 5 to 10 seconds.

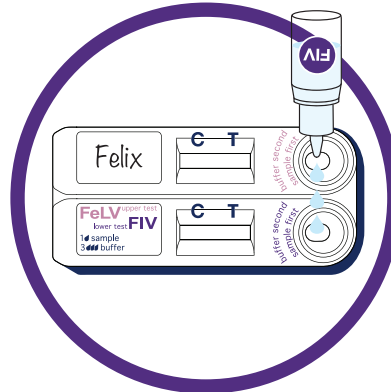

2
Add 3 drops of FIV Chase Buffer (purple cap) to the FIV sample well.

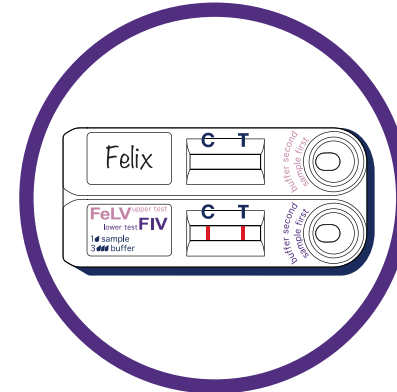

3
Read results within 10 minutes.
Positive Example



800.822.2947



www.abaxis.com



vetscan@abaxis.com

Singularly **Better.**



Abaxis, Inc.
Worldwide Headquarters

3240 Whipple Road
Union City, CA 94587
Tel 800 822 2947
Fax 510 441 6150

www.abaxis.com