Enzymatic Cleaner

(concentrate)

For veterinary *in vitro* Diagnostic use only.

INTENDED USE

A concentrated cleaning agent intended for use with the VetScan HMII, VetScan HM5 and VetScan HM2 hematology systems. Formulated to be aspirated into the instrument to clean components that come in contact with whole blood. The cleaner should be used for preventative maintenance at least once per month, during every reagent pack change or after performing approximately 100 tests. A cleaning should also be performed when the analyzer is repeatedly unable to obtain an acceptable blank run.

STORAGE CONDITIONS

Store at: 4–35 °C (40–95 °F)

WARNINGS

DO NOT INGEST. Keep away from food products. CAUTION: Eye irritant. Avoid contact with eyes and skin. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. Do not breathe spray. May cause sensitization by inhalation. Dispose according to applicable regulations.

REACTIVE INGREDIENTS

A concentrated solution of hydrolytic enzymes, surfactants and stabilizers. Biodegradable.

PROCEDURE

1. Hold uncapped bottle of Enzymatic Cleaner Concentrate upside down over an empty unused EDTA sample tube.
2. Squeeze 6-7 drops (approximately 300-350 µl) of Enzymatic Cleaner Concentrate into the sample tube. A minimum of 200 µl of cleaner must be in the tube, depending on the tube size that you have selected. A 1.3 ml tube is usually sufficient. You may have to use a larger volume of cleaner if you elect to use a larger volume tube.
3. Insert the tube into the proper adaptor and place onto the analyzer.
4. In the **Utilities** menu, select **Maintenance, Cleaning, Enzymatic Cleaning**.
5. Allow cleaning cycle to run.
6. Discard sample tube.
7. Running a control sample after a cleaning is recommended. See the analyzer User's Manual for more detailed instructions.

TECHNICAL ASSISTANCE:

For technical assistance or to place an order, call Abaxis technical support or customer service at 1-800-822-2947.

ORDERING INFORMATION

Abaxis VetScan Enzymatic Cleaner may be ordered using part no. 770-8010.

TRADEMARKS

VetScan is a trademark of Abaxis, Inc.

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VetScan ist ein Warenzeichen von Abaxis, Inc.
VetScan est une marque commerciale d'Abaxis, Inc.